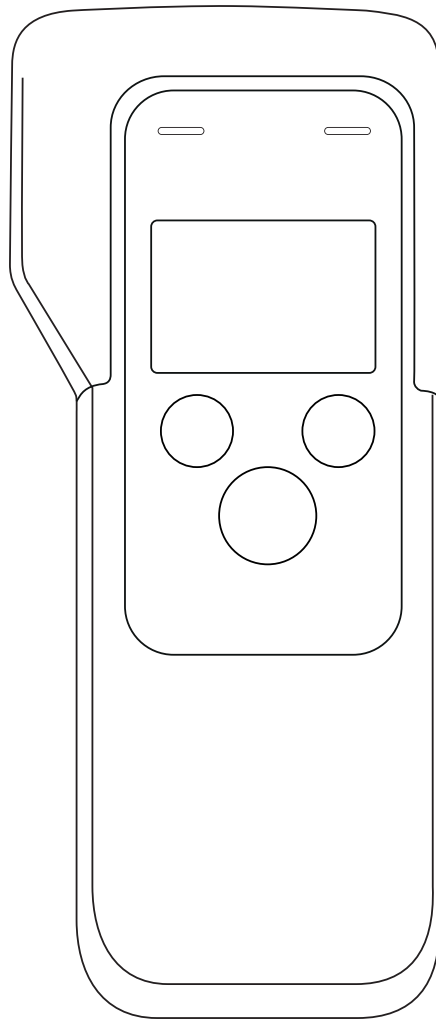
Instructions for use

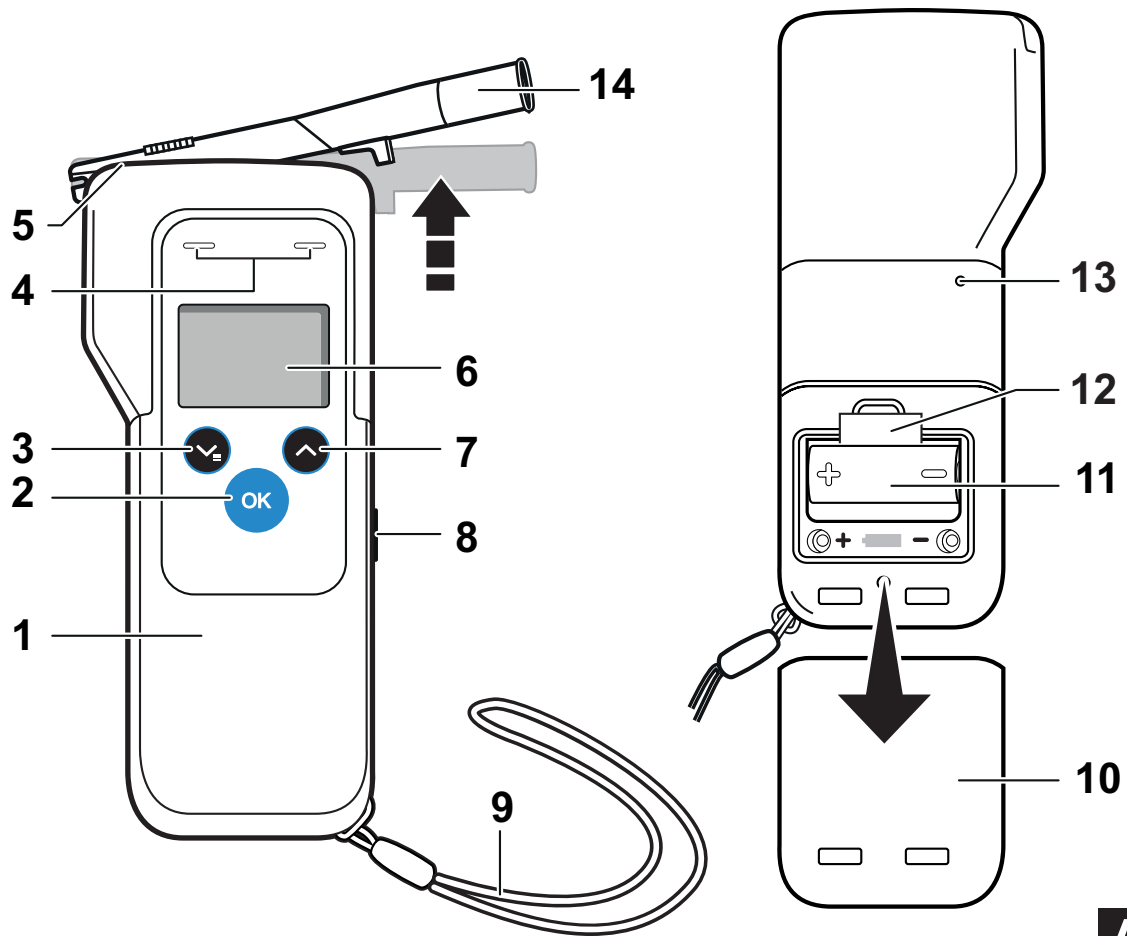
Dräger Alcotest 6000

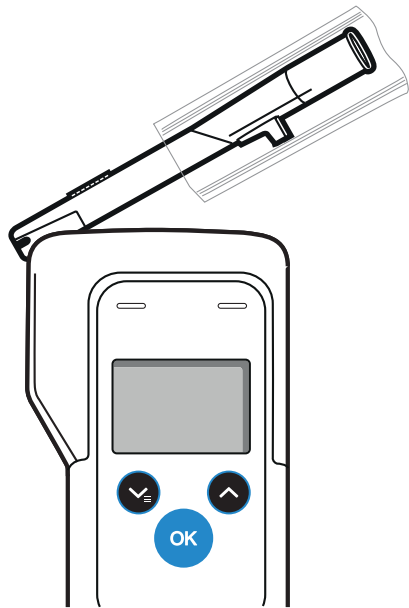
More languages:
draeger.com/ifu

de, en, fr, es, ca, ptBR, it, nl, da, fi, no, sv, et, lv, lt, pl, ru, hr, bs, sl,
sk, sr, cs, bg, uk, ro, hu, el, tr, ar, zh, zhTW, ja, ko

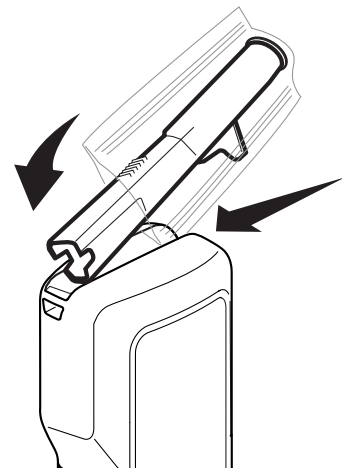
de	Gebrauchsanweisung	4
en	Instructions for use	10
fr	Notice d'utilisation	16
es	Instrucciones de uso	22
ca	Instruccions d'ús.....	28
ptBR	Instruções de uso	34
it	Istruzioni per l'uso	40
nl	Gebruiksaanwijzing	46
da	Brugsanvisning.....	52
fi	Käyttöohje	58
no	Bruksanvisning	64
sv	Bruksanvisning.....	70
et	Kasutusjuhend	76
lv	Lietošanas instrukcija	82
lt	Naudojimo instrukcija	88
pl	Instrukcja obsługi	94
ru	Руководство по эксплуатации.....	100
hr	Upute za uporabu.....	107
bs	Uputstvo za upotrebu	113
sl	Navodilo za uporabo	119
sk	Návod na použitie	125
sr	Uputstvo za korišćenje	131
cs	Návod k použití	137
bg	Ръководство за работа	143
uk	Настанова з експлуатації	149
ro	Instrucțiuni de utilizare	156
hu	Használati útmutató	162
el	Οδηγίες χρήσης.....	168
tr	Kullanım kılavuzu	175
ar	إرشادات الاستخدام	181
zh	使用说明书	186
zhTW	使用說明書	191
ja	取扱説明書	196
ko	사용지침서	202

A

B

C

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Contents

1	Safety-related information	11
2	Conventions in this document	11
2.1	Meaning of the warnings	11
2.2	Typographical conventions	11
3	Description	11
3.1	Product overview	11
3.2	Intended use	11
4	Use	11
4.1	Prerequisites for use	11
4.2	Initial operation	12
4.3	Taking an automatic breath sample	12
4.4	Taking a manual breath sample (optional)	12
4.5	Taking a passive breath sample (optional)	12
4.6	Removing the mouthpiece	12
4.7	Menu	12
4.8	Menu items	13
5	Printing the measurement results	13
6	Troubleshooting	13
6.1	Troubleshooting	13
7	Maintenance	14
7.1	Maintenance	14
7.2	Changing the battery	14
7.3	Cleaning	14
8	Storage	14
8.1	Storage	14
9	Disposal	14
10	Technical data	14
10.1	Technical data	14

1 Safety-related information

- Before using this product, carefully read these instructions for use and those of the associated products.
- Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.
- Do not dispose of the instructions for use. Ensure that they are stored and used appropriately by the user.
- Only trained and competent users are permitted to use this product.
- Do not use a faulty or incomplete product. Do not modify the product.
- Notify Dräger in the event of any component fault or failure.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommend a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Use only genuine Dräger spare parts and accessories. Otherwise, the proper functioning of the product may be impaired.

 These instructions for use can be downloaded in other languages in electronic format from the appropriate product page (www.draeger.com/ifu) or obtained free of charge as a printed copy from Dräger or the specialist shop.

- Only use chargers, power packs, batteries, or rechargeable batteries that Dräger has approved for this product.

2 Conventions in this document

2.1 Meaning of the warnings

The following warnings are used in this document to alert the user to possible hazards. A definition of the meaning of each warning is as follows:

Warning symbols	Signal word	Classification of warning
	WARNING	Indicates a potentially hazardous situation. If not avoided, it could result in death or serious injury.
	CAUTION	Indicates a potentially hazardous situation. If not avoided, it could result in physical injury. It may also be used to alert against unsafe practices.
	NOTICE	Indicates a potentially hazardous situation. If not avoided, it could result in damage to the product or environment.

2.2 Typographical conventions

Text Text in bold identifies labels on the device and text on the display.

- ▶ This triangle in warnings identifies the options available for avoiding the hazard.
- > The greater-than symbol identifies a navigation path in a menu.
-  This symbol indicates information that may make the use of this product easier.

3 Description

3.1 Product overview

See fold-out page

1	Breath alcohol testing device
2	 button (turning the device on/off; confirming entries)
3	 button (calling up the menu; setting a value)
4	LEDs
5	Mouthpiece receptacle (can be configured: right or left)
6	Display
7	 button (setting a value)
8	USB port (not for charging the device, for service personnel only)
9	Wrist strap
10	Battery compartment cover
11	Battery
12	Strap for changing the battery
13	Loudspeaker
14	Mouthpiece

3.2 Intended use

The Dräger Alcotest® 6000¹⁾ breath alcohol testing device is used to measure the breath alcohol concentration in a person's breath. This measurement is used to determine a person's blood alcohol level. Therapeutic measures should not be taken on the basis of the measurement.

4 Use

4.1 Prerequisites for use

WARNING

Small parts of the device may come loose and be swallowed.

- ▶ The device is not intended for children under the age of 3.

The device is adjusted in the factory before delivery. If the adjustment date is more than 6 months ago, then the device should be adjusted, e.g. by DrägerService.

- After the last alcohol consumption, wait for approx. 15 minutes.
- The wait time cannot be shortened by rinsing the mouth with water or a non-alcoholic drink.
- Residual mouth alcohol or aromatic drinks (e.g. fruit juice), alcoholic mouth sprays and medicines can falsify the measurement.

1) Alcotest® is a registered trademark of Dräger.

- Burping and vomiting can falsify the measurement result.

4.1.1 Before sampling

The test person must breathe regularly and calmly before sampling. The test person must be capable of providing the minimum breath volume required for sampling.

4.1.2 Attaching the wrist strap

Use the wrist strap to secure the device to your wrist.

1. Feed the fastening lug of the wrist strap through the eye on the device.
2. Pull the top part of the wrist strap through the fastening lug.

4.1.3 Preparing for use

1. Make sure a sufficiently charged battery has been correctly inserted.
2. Make sure a clean mouthpiece has been attached.
3. Make sure the device is securely held by using the wrist strap correctly.

4.1.4 Turning on the device

1. Press and hold the  button for approx. 1 second.
2. Wait until the display indicates that the device is ready to take a measurement.

4.1.5 Turning off the device

- Press and hold the  button for approx. 3 seconds.

 The device will turn itself off 4 minutes after last being operated.

4.2 Initial operation

When the device is turned on for the first time, a setup wizard appears. No setup wizard appears for pre-configured devices.

The following settings are made using the setup wizard:

- Language
- Date
- Time format
- Time

All settings can be subsequently changed via the menu.

 Loss of default settings

If the battery is removed for longer than 5 minutes, then the date and time will need to be reset.

4.3 Taking an automatic breath sample

 Use a new mouthpiece for every test person.

 Do not apply high concentrations of breath alcohol to the sensor, e.g. by rinsing the mouth with high-proof alcohol shortly before taking the measurement. Doing so would reduce the life span of the sensor.

1. Remove the mouthpiece (order no. 68 10 690 "Standard", 68 11 055 "with exhalation non-return valve") from the packaging.
For hygiene reasons, leave the mouth contact area protected by the packaging until the mouthpiece is firmly fixed on the device.

2. Place the mouthpiece in the mouthpiece receptacle at an angle, then press it into the mouthpiece receptacle (see fold-out page, figures B, C).

The ribbed marking (contact surface) of the mouthpiece points upwards.

⇒ The mouthpiece audibly clicks into place.

3. Completely remove the packaging.
4. Press and hold the  button for approx. 1 second until the startup screen appears.
An optional self-test is performed. Approx. 4 seconds later, the display shows the following:

- **READY**

⇒ The device is ready to perform a measurement.

5. Blow evenly and without interruption into the mouthpiece. A progress bar is displayed. An adequate breath flow is signalled by a continuous tone sounding. The continuous tone will stop with an audible click once an acceptable breath sample has been provided. After a few seconds (depending on the temperature of the device and the concentration measured), the measurement result is displayed with date and time.

4.3.1 Carrying out another measurement

1. Briefly press the  button to carry out a new measurement.
Once the sensor's recovery time has elapsed (see "Technical data", page 14), **READY** is shown.

4.4 Taking a manual breath sample (optional)

If the test person is not capable of providing the minimum volume required for sampling, sampling can be triggered manually.

1. Follow steps 1 to 3 of the automatic breath sample.
2. If the progress bar is shown after the breath sample has started, briefly press the  button.
When the results are displayed, they also show **MANUAL**.

4.5 Taking a passive breath sample (optional)

With passive sampling, sampling is triggered manually by the person using the device. Possible applications for this include testing the ambient air or a person's end-expiratory breath for the presence of alcohol.

1. Press and hold the  button for approx. 1 second until the startup screen appears.
2. Hold the prepared device **without** the funnel in the air to be analysed.
3. Briefly press the  button to start the measurement.
When the results are displayed, they also show **PASSIVE**.

4.6 Removing the mouthpiece

- Push the mouthpiece at the mounting spacer up and out of the mouthpiece receptacle and dispose of it (see fold-out page, figure A).

4.7 Menu

To access the menu:

- Press the  button.

To access the context menu:

- Press the  button.

To navigate to the menu or to configure the settings:

1. Press the  or  button.
2. Press the  button to confirm the function/input chosen.

To exit the menu:

- Choose the menu item **Back** in the sub-menus.
- Choose **Measuring** in the main menu.

4.8 Menu items

4.8.1 Measurement results

Shows all saved results with measurement unit, date and time, as well as sample number and the information as to whether the sample was taken manually or passively.

4.8.2 Infoscreen

Shows the part number and the serial number of the device, as well as the serial number of the EC sensor.

4.8.3 Next service

Displays the time until adjustment or service is due in days.

4.8.4 Date and time

Sets the date and time.

1. Change the digits using the  or  button.
2. Confirm with the  button.

4.8.5 Contrast

Adjusts the contrast.

1. Change the contrast setting using the  or  button.
2. Confirm with the  button.

4.8.6 Loudspeaker

Adjusts the volume.

1. Set the volume using the  or  button.
2. Confirm with the  button.

4.8.7 Language

Sets the language.

1. Select the language using the  or  button.
2. Confirm with the  button.

4.8.8 Measurement unit

Sets the measurement unit.

1. Select the measurement unit using the  or  button.
2. Confirm with the  button.

4.8.9 Software version

Displays the version number and part number of the software.

4.8.10 Signing in

For service personnel only.

1. Enter the four-digit code to open the service menu.

4.8.11 Bluetooth® (optional)

Turns the Bluetooth® feature on or off.

1. Confirm Bluetooth® **On** or **Off** with the  button.

 The device is prepared for printing and connecting to an app via Bluetooth®.

 An active Bluetooth® connection is displayed as a corresponding symbol on the display.

 Bluetooth® must not be activated in aeroplanes.

4.8.12 Turning off

Turns off the device.

 The device will turn itself off 4 minutes after last being operated.

1. Briefly press the  button.
- ✓ The device turns off.

5 Printing the measurement results

Bluetooth® can be optionally configured on the device. To print out the measurement results, use Bluetooth® to connect the device to the Dräger Mobile Printer Bluetooth® (part number: 3700421).

 Observe the instructions for use for the Dräger Mobile Printer Bluetooth®.

 Bluetooth® must not be activated in aeroplanes.

6 Troubleshooting

6.1 Troubleshooting

In the event of a device malfunction, an error message will be shown. The number which appears underneath the message is used for service functions. If the malfunction does not disappear after turning the device on and off several times, contact DrägerService.

Malfunction/Error	Cause	Remedy
Device cannot be turned on.	The battery is flat.	Change the battery.
Volume too small/sample invalid	The test person is blowing unevenly or not strongly enough.	Blow into the funnel with more force and without interruption.
Device outside temperature range	The device is too hot or too cold.	Allow the device to cool down or warm up.
††† (measuring range exceeded)	The measuring range has been exceeded. Residual mouth alcohol can lead to incorrect measured values.	At least 15 minutes wait time must have passed since alcohol was last consumed through the mouth. To repeat, press the  button.
Sample interruption	Irregular exhalation, abrupt ending or inhalation	Repeat the breath sample by pressing the  button

Malfunc-tion/Error	Cause	Remedy
Error number	Device error	Turn the device off and then back on. If the error persists, contact DrägerService or your retailer.

7 Maintenance

7.1 Maintenance

Have the device adjusted every six months as the measurement uncertainty increases over time due to drift effects. This adjustment must only be carried out by authorized persons, e.g. DrägerService.

Calibration certificates:

<https://www.draeger.com/productioncertificates>

7.2 Changing the battery

1. Turn off the device.
2. Push off the battery cover.
3. Replace the battery (battery type: 3 V CR123A; order no.: 45 43 808). Ensure that the polarity is correct when doing so.
4. Reattach the battery cover.
5. Check whether the time has been set correctly.

i If changing the battery takes more than 5 minutes, the time needs to be reset. Measurements cannot be performed if the time has not been set.

7.3 Cleaning

Clean the device using an ethanol-based cleaning agent.

To clean the device:

1. Turn off the device.
2. Remove the mouthpiece from the device.
3. Wipe the device using a disposable cloth moistened with cleaning agent. Do not apply the cleaning agent directly to the device.

i Make sure that no liquid enters the metal piece of the mouthpiece receptacle at the top of the device.

4. Carry out a passive sampling to ensure there is no residual ethanol inside the device and that the result displayed is **NO ALCOHOL**.
5. If **ALCOHOL** continues to be shown, repeat the passive sampling after a short break (10-15 minutes) until **NO ALCOHOL** is displayed.
6. If **ALCOHOL** continues to be shown, store the device for up to 24 hours to allow the cleaning agent to evaporate. Repeat the passive sampling after some time.

For information on disinfecting Alcotest devices, see document 9300698 at www.draeger.com/IFU.

8 Storage

8.1 Storage

⚠ WARNING

Damage to the device

Leaking battery acid can cause damage to the device.

- Remove the battery if the device is not going to be used for a prolonged period of time.

9 Disposal

This product must not be disposed of as household waste. This is indicated by the adjacent symbol.

You can return this product to Dräger free of charge. For information please contact the national marketing organizations or Dräger.

Batteries must not be disposed of as household waste. They are therefore marked with the adjacent symbol. Dispose of batteries at battery collection centres as specified by the applicable regulations.

10 Technical data

10.1 Technical data

Measuring principle

Electrochemical sensor

Measuring range

Breath alcohol concentration 0.00 to 2.50 mg/L
(Ethanol mass per breath volume at 34 °C and 1013 hPa)

Blood alcohol concentration 0.00 to 5.00 ‰
(Ethanol mass per blood volume or mass at 20 °C and 1013 hPa)

A conversion factor of 2100 is used to calculate the blood alcohol concentration using the breath alcohol concentration data (factory setting, country-specific conversion factors can be configured).

Measurement units

mg/L, µg/100 mL, µg%, µg/L, g/210 L, g/L ‰, %, mg/100 mL, mg%, mg/mL

The device meets the requirements of EN 15964. It can be configured in accordance with the standard. Display of measured values if configured in accordance with EN 15964:

Measured value	Display
0.0 mg/L to 0.029 mg/L	0.0 mg/L
0.03 mg/L to 2.5 mg/L	Measured value

Ambient conditions

During operation -5 °C to +50 °C
(23 °F to 122 °F)
10 to 100 % RH
(non-condensing)
600 to 1300 hPa

During storage	-20 °C to +60 °C (-4 °F to +140 °F)
	15 to 75 % RH (10 to 100 % RH <48 h)

Sampling

Breath volume	>1.2 L (default setting)
Breath flow	>10 L/min (default setting)
Breath duration	>1.6 s (default setting)

Measurement accuracy

Standard deviation of repeatability with ethanol standard

Up to 0.50 mg/L	0.008 mg/L or
≥0.50 mg/L	1.7 % of the measured value
Up to 1.00 ‰	0.017 ‰ or
≥1.00 ‰	1.7 % of the measured value
	depending on which value is higher

Sensitivity drift

Typically 0.4 % of the measured value per month

Wait time for measurement readiness following a previous measurement of:

0.00 mg/L: 2 s	0.00 ‰: 2 s
0.25 mg/L: 20 s	0.50 ‰: 20 s
0.50 mg/L: 40 s	1.00 ‰: 40 s
1.00 mg/L: 80 s	2.00 ‰: 80 s
>1.50 mg/L: 120 s	>3.00 ‰: 120 s

Adjustment

Recommendation: every 6 months (local deviations possible)

Dimensions

50/60 mm × 141 mm × 31 mm (W × H × D)

Weight:

approx. 150 g

Power supply

1 × 3 V CR123A

The battery is designed for up to 1500 measurements (depending on use and ambient conditions)

Protection class

IP 54

Interface

Micro-USB, 2.0 fullspeed; for service personnel only

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© Dräger Safety AG & Co. KGaA
Dräger Safety AG & Co. KGaA
Dräger Safety UK Ltd