


## Dräger DrugCheck® 3000 Alcohol and Drug Screening Devices

Use the Dräger DrugCheck® 3000 to find out within minutes if a person has drugs in his or her system. The compact and quick saliva-based drug test yields reliable results cheaply and easily. The device does not require electricity and can be used anywhere.

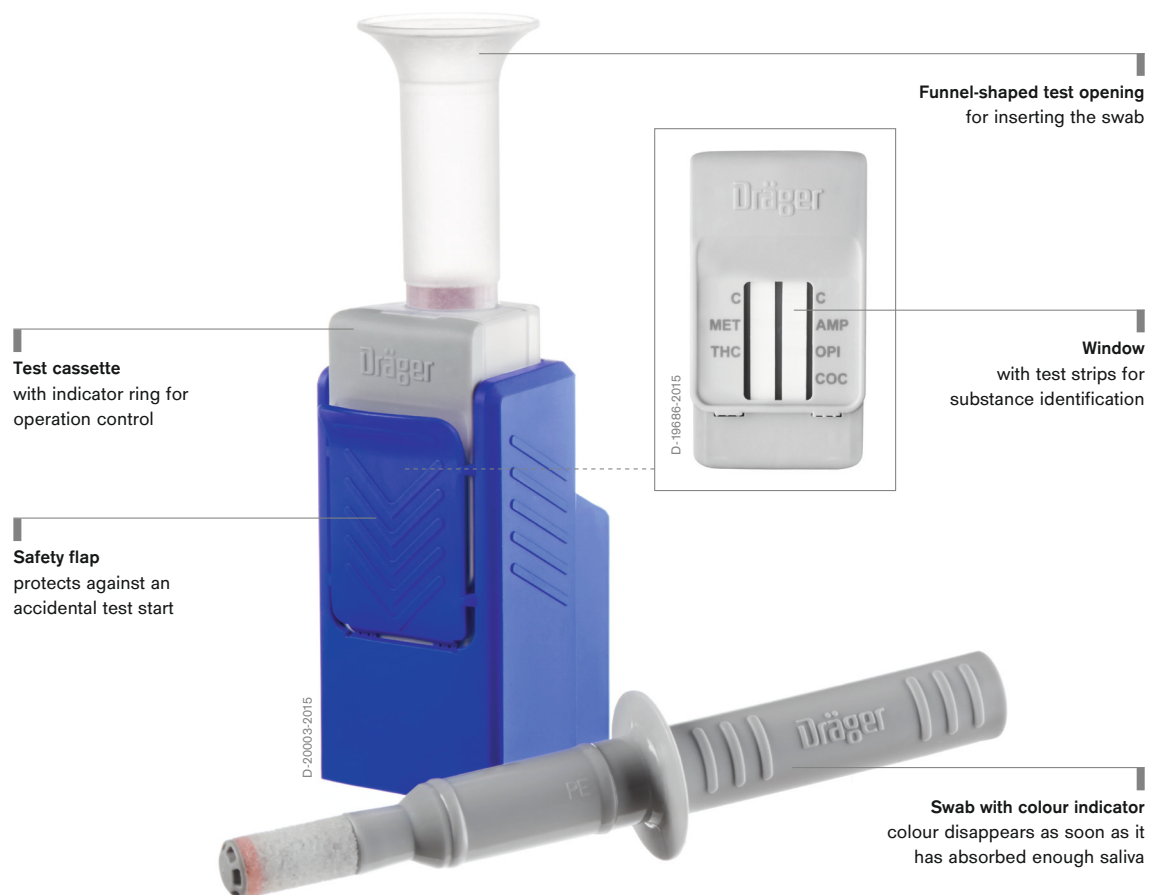

## Benefits

---

### Obtain samples easily and safely

The Dräger DrugCheck 3000 consists of two components: a swab for obtaining a saliva sample, and a test cassette for the analysis. The test cassette contains the buffer liquid and a window with two test strips, which display the control and test lines. The test itself is performed in three easy steps: Swab the saliva sample, shake the test kit, wait for a brief incubation period, and then start the test. A colour indicator on the swab disappears as soon as it has absorbed enough saliva for a test.

If the test result is negative, a line will appear alongside the respective substance class (drug). This means that none of the target substance was detected in the sample. If a line fails to appear next to one of the substance classes, then the result for this substance is positive. As soon as the control lines appear in the window, you can usually read the results after one to two minutes.

---

### Fast and sensitive analysis

Check individuals for up to five substance classes simultaneously with Dräger DrugCheck 3000: cocaine, opiates, amphetamine, methamphetamine, and cannabis (THC). Of all the substance classes listed, cannabis is the drug consumed most frequently and also the most difficult of all compounds to identify (THC). This is why the Dräger DrugCheck 3000 was optimised to detect THC, and now offers two measurement options: fast or sensitive. The fast mode displays a quantity of 40 ng/ml or more after just one minute. The sensitive mode permits detection of 15 ng/ml THC after three minutes.

---

### Unambiguous on-the-spot drug detection

Its compact design makes the DrugCheck 3000 easy to transport. It can be made ready for use quickly and easily, and is suitable for roadside checks and testing in safety-relevant workplaces\*. The test kit has no electrical parts, which makes it safe to use in hazardous areas.

---

### Additional benefits

Dräger has decades of experience in alcohol measuring equipment and drug detection methods. For the DrugCheck 3000, Dräger employed the testing principle of the proven Dräger DrugTest® 5000 system, which reliably detects even minute traces of THC. The disposable DrugCheck 3000 test kit cannot be manipulated and is hygienic to use.

\*The DrugCheck 3000 is sold only for law enforcement purposes in the USA.

## Related Products



D-54720-2012

---

### Dräger DrugTest® 5000

No pipetting, no drips, no timing: Collecting an oral fluid sample is quick and easy with the Dräger DrugTest 5000. The collected sample can then be analysed immediately for accurate results on the spot.

## Technical Data

### Test for detecting drugs in saliva

Dimensions (W x H x D)	32 x 111 x 57 mm
Weight	< 30 g (0,07 lb)
Operating range	Operation: 0 °C to 30 °C (32 °F to 86 °F) at 5% to 95% relative humidity
Storage / transportation	4 °C to 25 °C (39 °F to 77 °F)
Time of measurement	fast measurement: analysis < 5 minutes sensitive measurement: analysis < 7 minutes
Selecting the measurement mode	Dependent on the selected pre-incubation period and desired THC cut-off: fast: at 1 minute and 40 ng/ml sensitive: at 3 minutes and 15 ng/ml
Licenses / standards	Licensed as a medical product within the EU in accordance with Directive 98/79/EC on in vitro diagnostic medical devices. Outside of the European Union for forensic use only (Non-IVD).

## Ordering Information

DrugCheck® 3000	Description	Ordering no.
Test for detecting drugs in saliva		
DrugCheck® 3000 STK 5 IVD	Packaging unit 20 kits	83 25 500
DrugCheck® 3000 STK 5 Non-IVD	Packaging unit 20 kits	83 25 580

The Dräger DrugCheck® 3000 detects the following substances:

- Cocaine (COC)
- Opiates (OPI)
- Cannabis (THC)
- Amphetamine (AMP)
- Methamphetamine (MAMP)

#### CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
[www.draeger.com](http://www.draeger.com)

#### MANUFACTURER

Dräger Safety AG & Co. KGaA  
Revalstraße 1  
23560 Lübeck, Germany

#### REGION EUROPE CENTRAL AND EUROPE NORTH

Dräger Safety AG & Co. KGaA  
Revalstraße 1  
23560 Lübeck, Germany  
Tel +49 461 882 0  
Fax +49 461 882 2080  
[info@draeger.com](mailto:info@draeger.com)

#### REGION EUROPE SOUTH

Dräger Safety France SAS  
25, rue Georges Besse  
92182 Antony Cedex  
Tel +33 1 46 11 56 00  
Fax +33 1 40 96 97 20  
[europe.south@draeger.com](mailto:europe.south@draeger.com)

#### REGION MIDDLE EAST, AFRICA

Dräger Safety AG & Co. KGaA  
Branch Office  
P.O. Box 505108  
Dubai, United Arab Emirates  
Tel +971 4 4294 600  
Fax +971 4 4294 699  
[contactuae@draeger.com](mailto:contactuae@draeger.com)

#### REGION ASIA PACIFIC

Draeger Safety Asia Pte Ltd  
25 International Business Park  
#04-20/21 German Centre  
Singapore 609916  
Tel +65 6308 9400  
Fax +65 6308 9401  
[asia.pacific@draeger.com](mailto:asia.pacific@draeger.com)

#### REGION CENTRAL AND SOUTH AMERICA

Dräger Panama S. de R.L.  
Complejo Business Park,  
V tower, 10th floor  
Panama City  
Tel +507 377 9100  
Fax +507 377 9130  
[contactcsa@draeger.com](mailto:contactcsa@draeger.com)

Locate your Regional  
Sales Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)

